# Eye-Lotion

Balanced salt solution

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# Eye-Lotion

Balanced salt solution

Sterile, physiological irrigation solution  
For ophthalmic procedures

**Composition**

1 ml solution contains:

sodium chloride	6.40 mg
potassium chloride	0.75 mg
calcium chloride 2 H <sub>2</sub> O	0.48 mg
magnesium chloride 6 H <sub>2</sub> O	0.30 mg
sodium acetate 3 H <sub>2</sub> O	3.90 mg
sodium citrate 2 H <sub>2</sub> O	1.70 mg

Excipients:

water for injections (sodium hydroxide or hydrochloric acid up to pH 7.2)  
No buffered solution (according to division EN ISO 16671)

**Uses**

- irrigation of the anterior chamber during cataract operations and other intraocular procedures
- rinsing out alpha-chymotrypsin after application of enzymatic zonulolysis
- intraoperative irrigation of the anterior chamber of the eye in the case of bleeding into the anterior chamber
- irrigation of the cornea during strabotomy operations and other extraocular procedures
- irrigation of the conjunctiva after fluorescein instillation
- irrigation of the anterior part of the eye and irrigation of the nasolacrimal ducts in the event of burns and acid burns
- removal of foreign bodies or dead cells
- keeping the cornea and conjunctiva moist during laser treatment

**Side-effects**

Where there is pre-existing pathological damage to the corneal endothelium, any intraocular intervention – including intraocular irrigation – can lead to bullous keratopathy which is characterised by corneal oedema or clouding. After the surgical procedure, inflammatory processes, including endophthalmitis, can occur. No connection with the use of Eye-Lotion has yet been demonstrated.

**Dosage recommendations**

The dosage is determined by the particular circumstances dictated by the operative, therapeutic or diagnostic procedure and should be decided by the doctor on a case by case basis.

**Method of administration**

To irrigate the eye for the purpose of surgical procedures as well as diagnostic and therapeutic measures that require irrigation or moisture retention.

An infusion set 'P' (pressure-resistant) with built-in air vent should be used. Bottles do not have added venting, which is why the air vent on the infusion set must be opened. In the case of bags, the valve can remain closed (bag collapses). The directions for use from the manufacturer of the infusion set must always be followed. The infusion set should be fitted onto the bottle/bag as follows (aseptic conditions):

1. Remove the protective cap from the adapter on the bottle/bag.
2. Disinfect the rubber stopper on the bottle/bag.
3. Push the puncture spike of the infusion set into the rubber stopper.

Note: The fluid should already be flowing before use, in order to ensure there is no air in the tubing.

The period of use is limited by the duration of the procedure.

References in the literature reveal that the osmolality of the aqueous humour is higher in diabetics and intraoperative lens changes can arise. Since Eye-Lotion is iso-osmotic with normal aqueous humour, it should be used with caution in diabetics undergoing a vitrectomy. In these cases it is advisable to match the osmolality to that of the aqueous humour.

**Special instructions**

- Use only for eye irrigation and for eye surgery, not for injection or infusion.
- Only use solution if it is completely clear and no minute particles are visible.
- Before use, check that the container is intact. Do not use, if sterile packaging was damaged.
- Use the solution immediately after opening the container.
- Eye-Lotion contains no preservatives; therefore do not store opened containers for further use. The medical device is intended solely for single use and is not suitable for reprocessing.
- Do not use Eye-Lotion as solvent for other medicinal products.
- The expiry date relates to an unopened product in its closed and properly stored packaging. Do not use beyond the expiry date given on the pack.
- Keep out of the reach of children.

**Storage**

glass bottle: Store at 5 – 25 °C  
PP bottle and PP bag: no special requirements

**Presentation**

glass bottle with 500 ml;  
PP bottle with 250 ml and 500 ml;  
PP bag with 500 ml  
Not all pack sizes may be marketed.

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**Explanation of the symbols**

	Consult instructions for use
	Do not use if package is damaged
	Do not re-use
	Sterilized using steam
	Article number
	Batch code
	Use-by date
	Manufacturer

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Serumwerk Bernburg Vertriebs GmbH